

# CUSTOMIZABLE LIFT SUPPORT

Offered **ONLY** by Golden

## WHAT IS STRAIGHT LIFT?

Straight Lift Positioning was designed to allow for **safer transfer options** by lifting the chair straight up instead of tilting forward when getting in and out. This vertical lift motion helps **maintain stability and control**, providing a more secure transition to a standing or seated position.

## WHO NEEDS IT & WHY?

Available on all Golden chairs, this innovative feature supports **greater independence** and **confidence for those with mobility challenges**, while promoting joint health, safety, and **overall wellness** through smoother, more natural transitions.

## 3 TYPES OF SAFETY TRANSFERS



### FULL LIFT

- Gently raises the user to a full standing position
- Adds extra support for those with limited mobility
- Ideal for recovery from abdominal, hip or knee surgery, or managing chronic mobility challenges



### MID LIFT\*

- Provides a gradual straight lift before transitioning for an easy transfer
- Promotes comfort with a gentler seat angle, ideal for those seeking a more supported and relaxed position
- Makes returning to a seated position easy



### STRAIGHT LIFT

- Exclusive feature not found on other recliners
- Enables safe transfers with the assistance of a caregiver
- Helps smooth transitions to and from power and manual wheelchairs
- Provides relief for those with back pain
- Provides comfort by allowing taller users to adjust seat height

*\*Note: Mid Lift is currently available on all models except 5-Zone chairs with Twilight Tilt Technology*



[LEARN MORE](#)

# HOW TO CONVERT CHAIR


Offered **ONLY** by Golden



WATCH TUTORIAL VIDEO



1. Unplug the transformer from the wall outlet & unplug battery back up from chair.
2. Gently push the chair over onto the footrest, turn it upside down, or gently push it over onto its side, opposite of the hand control.
3. Remove Clevis Pin & Hairpin that retain lower bar on rearward sides of the lift frame (*scan QR Code for a closer look*).
4. Move bar up and align holes. Middle hole is for Mid Lift and top hole is for Straight Lift.
5. Insert removed Clevis Pin into the new location & reinstall hairpin.
6. Repeat steps 3-5 for the opposite side.

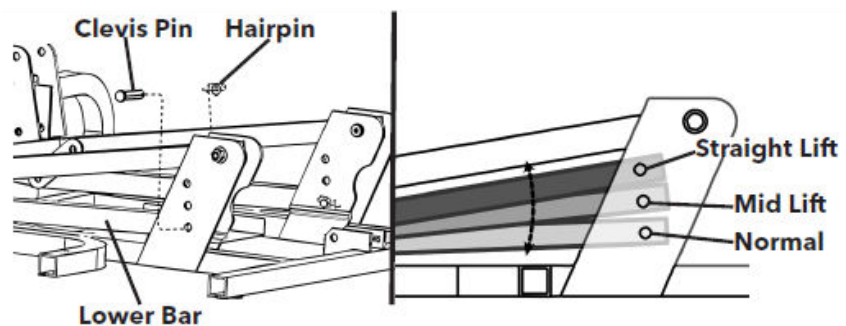

VIEW OWNER'S MANUAL

